


Simple Medical (Shenzhen) Limited

Instruction For Use
SIMPLE Walker B2
Model: 61006 Size: XS, S, M, L, XL

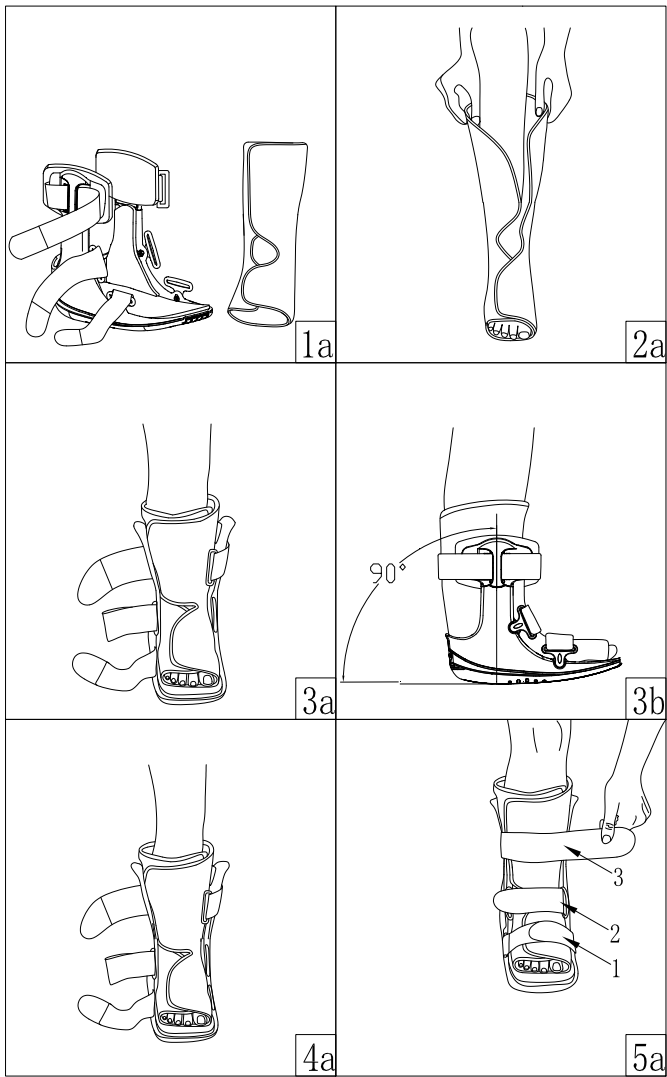

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.

Product overview
1)Product name: SIMPLE Walker B2
2)Product model: 61006
3)Product specification: XS, S, M, L, XL
4)Product Structure:
The product is mainly composed of base, Strut, straps, lining etc
5)Operating principle:
The main body of the product is provided with support, anti-slip soft sole and shock absorption system. The ankle and calf are properly protected through the support. Through software and shock absorption system, patients have more comfort in their legs during exercise. The shoe body big body and the calf combination, causes the patient leg to obtain the perfect protection. The breathable soft shoe pad enables the product to provide comprehensive protection for the feet before and after use, to prevent secondary injury, and to provide soft contact between the calf and the fixed support to make walking more comfortable. Can give the patient's foot a very good buffer, help the patient get a good recovery.
Scope of application:
External use, provides support for foot fracture, foot ligament injury fixation, foot sole and toe injury fixation, mild and moderate foot thumb valgus, plantar pain, postoperative rehabilitation
Product features and performance description
1) Product features and storage conditions
【 Product features 】 Low profile rocker sole, Shock-abosobing technology, High strength & light weight, Mechanical strap locking system.
【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature
2)Using the operating 【Application Instructions】
Note: Do not contact the liner directly with the skin.
*Please purchase and use the product under the guidance of your doctor or professional.
STEP 1: Release the nylon buckle and remove the internal gasket (Fig1a).
STEP 2: place the foot in the pad, heel close to the back of the pad, wrap the foot and leg. Make sure the legs and feet are close to the gasket (Fig 2a).
STEP 3: gently expand the side pillar and place the calf in the bracket while the pull strip is intact. The ankle should be concentrated on the pillar, with the foot and ankle at 90 °(Fig3a~3b).
STEP 4: a After correct alignment, adjust the straps and gaskets to know the comfortable position (Fig 4a)
STEP 5: Fasten the strap 1-2-3 from the bottom to the top of the toe, pay attention to the hole position of the strap and the column position on the support frame can not be separated, fasten the strap not too tight. . As the swelling pattern changes over the day, it may be necessary to adjust the tightness of the strap for comfort(Fig 5)
STEP 6: To remove, simply release the tape and pad and pull the heel and ankle forward to reapply the foot and ankle into the gasket and

reconnect the buckle (Fig 5a)
IMPORTANT:
1) Do not contact the liner directly with the skin.
2) Warnings and Over tightening or improper bandaging may cause severe skin irritation, especially for diabetic neuropathy (a diabetic neurological condition). Always check for "hot spot" and skin irritation on your skin. If you find any skin changes or feel uncomfortable, loosen the tightness of the dressing. Do not use the device if it is not felt due to post-operative anesthesia, nerve, skin, or other conditions.
Maintenance:
The software part can be washed with a neutral detergent and a 30 & deg; C sailor and air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches.
The rigid part can be scrubbed with wet cloth and dried naturally.
Warning:
If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.
*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.
Basic product environment
1) Ambient temperature: -15 ~ 60 °C
2) Relative humidity: 80%
Product material
1) Base/ strut: polypropylene+GF
2) Straps: nylon
3)Pad: nylon / polyurethane / polyester (outer net / foam / inner mesh)
4)Shoeoutsole/shoe inner sole: thermoplastic elastomer / polypropylene/EVA
Contraindication
! Use with caution for unstable fractures;
! Use with caution in osteoporosis;
! Use with caution for cognitive impairment;
! Use with caution for bony ankylosis;
Storage, transportation and expiry date of products
1)Product is valid: five years
2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
3)Transportation: the product shall be dust-proof, moisture-proof and pollution-proof during storage and transportation
Disposal of wastes
Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description
Manufacture Date LOT Batch Code
MD medical device
Hand wash only
Protect from sunlight
Do not use when the packing is damaged
Warnings and Precaution Reference instructions

Accessories list
Instructions ----- 1 copy
Pad ----- 3 pcs
Selection

Item	61006	Short			
Size	XS	S	M	L	XL
Insole Length(cm)	22	24.5	27.5	30.5	32.5
Outsole Length(cm)	24	26.5	30.5	32.5	34
Men's Shoe Size US	2~4	4.5~7	7.5~10.5	10.5~12.5	12.5+
Women's Shoe Size US	3.5~5.5	6~8	8.5~11.5	11.5~13.5	13.5+
Shoe Size EU	30~34	34~38	38~44	44~47	47~50

Warranty
Simple Medical (Shenzhen) Limited warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Simple Medical (Shenzhen) Limited in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street,Longhua District, Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: service@simplegroup.net
EC REP SUNGO Europe B.V.

Fascinatïo Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands
Tel : +31(0)10-3034500
*Please refer to the Instructions before usage.
CE
61006 V0.2